

DER MAP

AN EASY WAY TO MAP
VITILIGO

INSTRUCTIONS FOR USE

WARNINGS

1. SWITCH ON/OFF
2. SELECT YOUR HAND DIMENSION
3. INSERT THE MASK
4. PLACE THE SHEET
5. INSERT YOUR HAND
6. DRAW THE SPOT!
7. MAINTENANCE

WARNINGS



PREVENT THE ELECTRONIC CIRCUITS FROM GETTING WET. DO NOT ATTEMPT TO OPEN THE DEVICE IN WAYS OTHER THAN THOSE INDICATED IN THE MANUAL. DO NOT DROP, KNOCK OR SHAKE THE DEVICE. DO NOT USE CORROSIVE CHEMICALS, SOLVENTS OR AGGRESSIVE DETERGENTS TO CLEAN THE DEVICE.

1. SWITCH ON/OFF




TO SWITCH THE DEVICE ON OR OFF, PRESS THE SWITCH ON THE BOTTOM.

2. SELECT YOUR HAND DIMENSION




USING THE DIFFERENT SIZES PROVIDED, IDENTIFY THE ONE THAT BEST SUITS YOUR HAND.

3. INSERT THE MASK

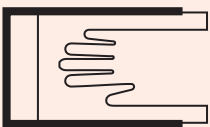


ONCE YOU HAVE IDENTIFIED THE SIZE, TAKE THE CORRESPONDING MASK AND INSERT IT INTO THE APPROPRIATE GUIDES OF THE WOODEN STRUCTURE.

4. PLACE THE SHEET

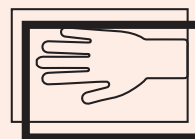

SELECT THE SAME SIZE SHEET AND PLACE IT ON TOP OF THE STRUCTURE BY CENTERING THE HOLES ON THE APPROPRIATE POSTS.

5. INSERT YOUR HAND

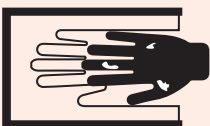

SLIDE YOUR HAND INTO THE MASK UNTIL IT STOPS.

6. DRAW THE SPOT!



TAKE THE MARKER AND DRAW THE SPOTS ON THE SHEET AS YOU SEE THEM. WHEN YOU ARE FINISHED REMOVE THE SHEET AND TURN OFF THE DEVICE.

7. MAINTENANCE

USE A SOFT CLOTH FOR CLEANING PANELS.
TO CHANGE THE BATTERIES REMOVE THE FEETS,
UNSCREW THE SCREWS AND LIFT THE PANEL.